

Normetanephrine ELISA kit

Fast – Urine Samples

Ref: BA-E-8500

This Normetanephrine ELISA kit enables to quantify normetanephrine in urine samples within 2 hours. Intended for *in vitro* diagnostics & research use, this quantitative immunoassay demonstrated good correlation with HPLC.

Sample type	25µl urine
Samples/kit	40 samples in duplicate
Sensitivity	23ng/ml
Range	30 - 3 000ng/ml
Assay time	Sample hydrolysis and acetylation (45 min), ELISA (1h)
Reactivity	Reacts with all species
Citations	Cited in 1 paper

INFORMATIONS

Product overview

Product name	Normetanephine ELISA kit – Urine- Fast-Track
Description	<i>In vitro</i> diagnostics & Research-use enzyme immunoassay (ELISA) for the fast determination of normetanephine (normetadrenaline) in urine samples
Labels	IVD (EU only) or RUO, CE
Format	96-well plate
Samples	Urine
Sample volume	25µL
Reactivity	Reacts with all species
Standard range	0 / 30 – 3 000ng/ml
Sensitivity	23ng/ml
Specificity	No significant cross-reactivity was observed with Normetanephine analogs such as metanephine, 3-methoxytyramine, Adrenaline, Noradrenaline, Dopamine, Vanillic mandelic acid, L-Dopa, Homovanillic acid, L-Tyrosine and Tyramine
Assay time	Sample hydrolysis and acetylation 45 min and ELISA 1h
Storage	Store at 2-8°C for to 6 months.

Datasheets[Instructions for use](#), [Material safety datasheet](#)

PROTOCOLS

Sample collection & storage	Spontaneous or 24-hours urine, collected in a bottle containing 10-15 mL of 6 M HCl. Storage at -20°C. for long periods (up to 6 months)
--	--

Sample preparation	Sample hydrolysis (30 min) and acetylation (15min)
---------------------------	--

ELISA	Normetanephrine antiserum incubation (30 min), revelation and read steps (30h)
--------------	--

Detailed protocol	Download instructions for use
--------------------------	---


REFERENCES

Product citation

Product pictures



Urine Normetanephrine ELISA

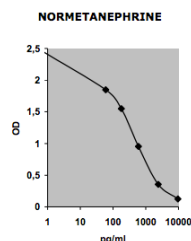

Urinary Normetanephrine ELISA: Standard curve

Standard curve - to be generated every time the assay is performed.

Contact information

Immusmol
229 Cours de l'Argonne
33 000 Bordeaux - France
Tel: +33 (0) 5 6431 1170
www.immusmol.com

To order, review, ask for technical support, visit product page at:

<https://www.immusmol.com/shop/normetanephrine-elisa-kit-urine-samples/>